

UK NEQAS Manufacturer's Proforma Template

Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

	
Title	Instructions for running the UK NEQAS FB EQA Samples on the Sysmex XN, XNL, XQ and XR series analysers
Analysis Instructions	UK NEQAS Full blood count survey samples do not contain material suitable for the counting of Platelets by optical or fluorescent methods on Sysmex haematology instruments. Please process these samples in CBC or CBC+DIFF modes to ensure that you only report impedance platelet counts on Sysmex XN, XNL, XQ and XR series analysers
Additional Notes:	Examples of manufacturers letters can be viewed via https://www.ukneqash.org/documents/1
Contact Information	Sysmex Customer Support Haem.support@sysmex.co.uk 0333 320 3466
Authorised by	Matthew Paluszkiewicz Haematology Product Manager
Document Control	- Reference details/ number: ECD 530 - Version number: 3.0 - Authorisation date: 10/11/2025