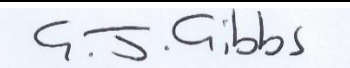


UK NEQAS Manufacturer's Proforma

Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

		
Title	White Cell Count Reporting on Siemens ADVIA Haematology Systems	Guidance on WBC FB and DL parameter selection for ADVIA 120, 2120, and 2120i Analysers
Analysis Instructions	Because the samples for the full blood count are prepared differently than those for the automated differential leucocyte count, they do not behave the same way when processed by the ADVIA 120, ADVIA 2120 or ADVIA 2120i. The white cell count results should therefore be reported as follows: ▪ NEQAS Full Blood Count samples: Baso WBC ▪ NEQAS Automated Differential Leucocyte Count: Perox WBC	
Additional Notes:	~	
Contact Information	Technical Support Team Email Address: haematology.team@siemens-healthineers.com Telephone Number: 0345 600 1955	
Authorised by	Graham Gibbs 	
Document Control	○ Reference details/ number: ECD62 ○ Version number: 2.0 ○ Authorisation date: 17/10/2025	