



Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers



CD H5027 v1 Date: 26.08.2025 Page **1** of **5**

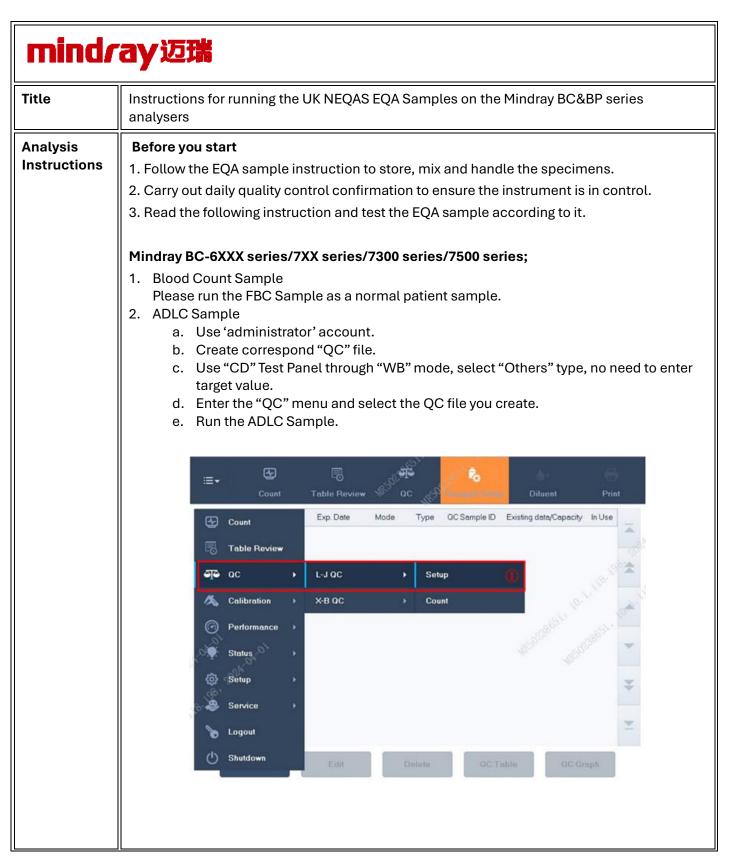

Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

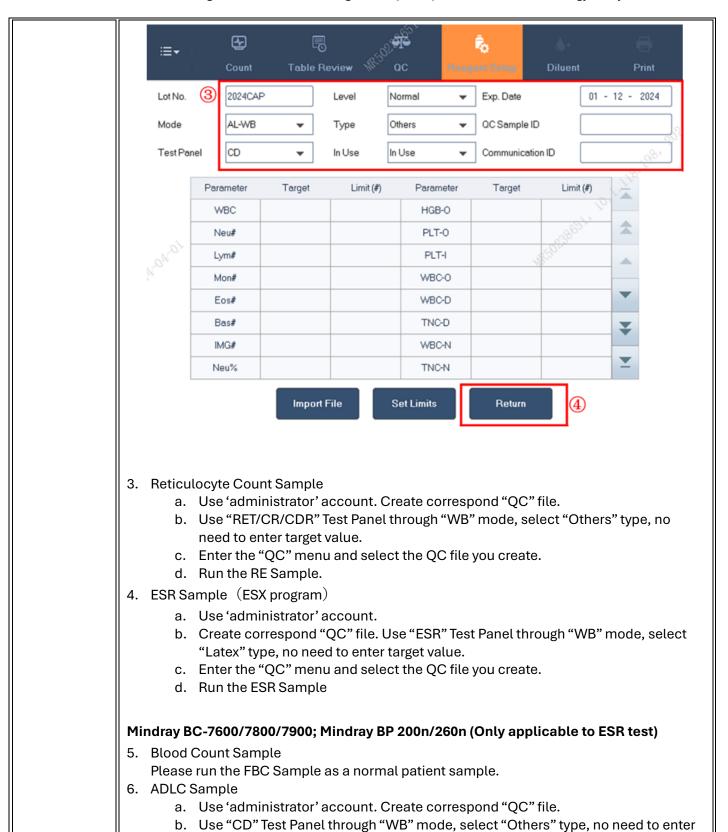


CD H5027 v1 Date: 26.08.2025 Page 2 of 5

target value.




Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

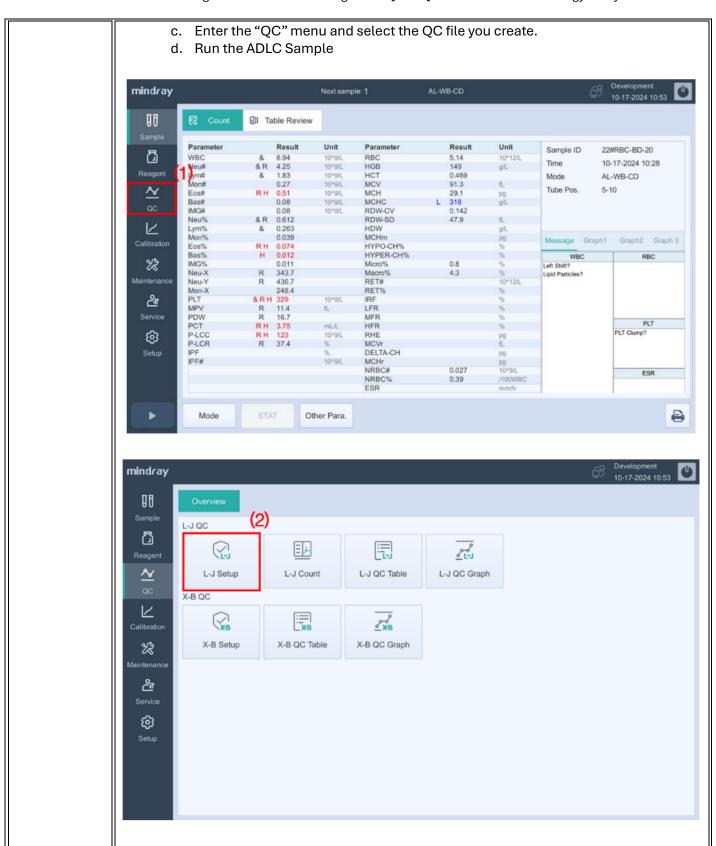

CD H5027 v1 Date: 26.08.2025 Page **3** of **5**



Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

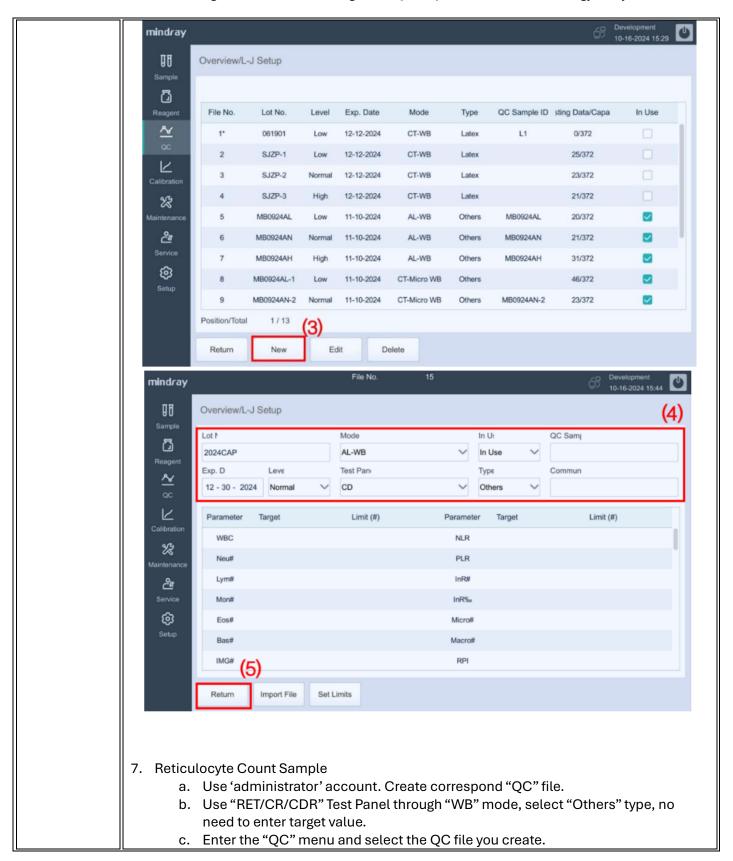

CD H5027 v1 Date: 26.08.2025 Page **4** of **5**



Guidance for Providing Instructions on Running UK NEQAS EQA Material on Haematology Analysers

	d. Run the RE Sample
	8. ESR Sample (ESX program)
	 a. Use 'administrator' account. Create correspond "QC" file. b. Use "ESR" Test Panel through "WB" mode, select "Latex" type, no need to enter target value. c. Enter the "QC" menu and select the QC file you create. d. Run the ESR Sample.
Additional	Attach relevant diagrams, visual guides, or screenshots to support the instructions.
Notes:	Note reference to any attached documents
Contact Information	Examples of manufacturers letters can be viewed via https://www.ukneqash.org/documents/1 Technical Support Team: Mindray Traceability and Reference Research Department Email Address: shixiaoyong@mindray.com ; caohaiyan_a@mindray.com Telephone Number: +86 755 81885140
Authorised by	Name: XIAOYONG SHI Signature: SHI XIAO YONG
Document Control	 ECD 688 Version number: 1.0 Authorisation date: 2025/8/12

CD H5027 v1 Date: 26.08.2025 Page **5** of **5**